## Indications for use

Postoperative care of surgery (cranial cruciate ligaments), traumatology, osteoarthritis crisis.  
Analgesic, anti-oedematous, anti-haemorrhagic effects.

## Contraindications

Skin sensitivity, hypothermia, peripheral vascular disorders (agglutinin disease cold) and diabetes mellitus.

## Recommendations for use

For more efficiency, the dog must remain lying down during the session. When setting up the device, be sure to secure it with the straps.

## Risks of use and precautions for use

This device must be used according to the prescription of your veterinary.

Do not exceed 20 minutes per session.

The manufacturer makes no warranty that the use of this product will not prevent injury. It is imperative to check the completeness and integrity of the elements before use. Do not use if damaged.

Cold pack should never be in direct contact with the scar or the skin. Systematically use the hypoallergenic lycra supplied. Cold pack should be soft and frost free before use.

Do not heat or warm cold pack.

Respect the number of pressures indicated to inflate the pack. In case of unusual agitation of the animal (discomfort due to cold), remove and deflate the pack immediately.

Do not pierce the cold pack, nor the manual pump and its connection pipe.

In the event of gel leakage: handle the device with gloves, do not ingest the gel, do not pour the gel into a drainage system connected to the sewers, or into a watercourse.

Do not throw the device in nature or with household waste. Disposal is done with special industrial waste, in the recycling center.

In case of direct contact of the gel with the skin, rinse immediately and abundantly with cold water.

Contact your doctor and/or your veterinary if irritation persists.

Keep out of the reach of children and people unfit to use this device.

## Cleaning

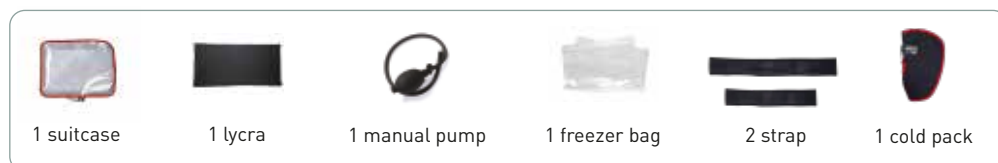
Cold pack and lycra are hand washable with mild soap. Do not machine wash and never use bleach. Do not immerse in a disinfectant detergent bath.

If necessary, spray a disinfectant detergent solution (ANIOS disinfectant detergent for high surfaces).


Always place the tap in position 2. Leave to air dry and lay flat.

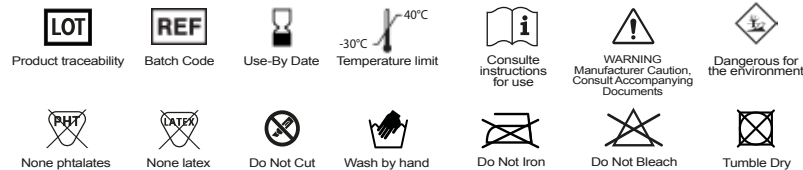For more information, contact Orthonov or visit the website [www.orthonov.com](http://www.orthonov.com)

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